

schülke →



Ready to use disinfectant wipes
impregnated with an alcoholic solution.

mikroloid® AF wipes

PRAXISDIENST
Medical Supplies since 1953
Order here!

Our Plus

- ready to use, impregnated disinfection wipes
- very broad biocidal activity within the shortest possible time
- dries rapidly
- wide range of wipe sizes and packaging

Application areas

Alcohol-based rapid disinfection of medical devices in all areas and all kinds of wipeable surfaces with an increased risk of infection and where short contact times are required, e.g.:

- Patient nearby surfaces
- Patient treatment units
- Examination couches
- Operating tables and nearby work surfaces
- Surfaces of medical equipment

Instructions for use

Thoroughly wipe the surface with the impregnated wipe and allow to take effect. Make sure to wet surfaces completely and keep them wet for the whole exposure time. Make sure that all visible dirt is removed prior to disinfection. When opening the lid and inserting the first cloth, make sure that hygiene requirements are maintained. Make sure to close the lid tightly after use. Do not treat surfaces that are sensitive to alcohol, e.g. acrylic glass. Not suitable for the disinfection of invasive medical devices. schülke guarantees, if used in the correct way, a shelf life of 3 months after the package has been opened.

Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
MRSA	ready-to-use	1 min.
mycobactericidal <i>in accordance with VAH Guidelines with short contact times - dirty conditions</i>	ready-to-use	1 min.
fungicidal	ready-to-use	2 min.
levurocidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
enveloped viruses (incl. HIV, HBV, HCV) <i>in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline</i>	ready-to-use	30 sec.
Norovirus EN14476	ready-to-use	1 min.
Rotavirus	ready-to-use	30 sec.
Polyoma SV40 <i>clean conditions</i>	ready-to-use	10 min.
Adenovirus (type 5) <i>in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline</i>	ready-to-use	5 min.

Certificates

- VAH certificate



mikrozid® AF wipes

Product data

Composition:

100g agent solution contain the following active ingredients:
25 g Ethanol (94 %), 35 g Propan-1-ol

Chemical-physical data

Color	colourless
Density	ca. 0,89 g/cm ³ / 20 °C
Flash point	27 °C / Method: DIN 51755 Part 1
Form	liquid
pH	Not applicable
Viscosity, dynamic	not determined

Special advice

Use disinfectants safely. Always read the label and product information before use.

The product is highly compatible with metals and synthetics (except acrylic glass and alcohol-sensitive varnishes). Please follow carefully any national fire and explosion protection guidelines when using alcohol-based disinfectants. Only treat small surfaces. The wiping range depends on the room temperature and on the structure of the surface to be disinfected. More information available on request.

Refilling of the tin

First disinfect hands and wear new protective gloves. During the refilling process, please ensure that the gloves are not contaminated by other activities. On every refill, reprocess the lid closure. In reprocessing, subject the lid closure to disinfectant cleaning with an alcohol based surface disinfectant.

Following sizes are available:

- mikrozid® AF wipes (10 x 150 wipes 14 x 18 cm) Surface coverage*: approx. 0.3 m²
- mikrozid® AF wipes Refill (20 x 150 wipes 14 x 18 cm) Surface coverage*: approx. 0.3 m²
- mikrozid® AF wipes *Jumbo* (10 x 200 wipes 20 x 27 cm) Surface coverage*: approx. 0.5 m²
- mikrozid® AF wipes *Jumbo* Refill (12 x 200 wipes 20 x 27 cm) Surface coverage*: approx. 0.5 m²
- mikrozid® AF wipes *premium* (12 x 50 wipes 20 x 20 cm) Surface coverage*: approx. 0.5 m²

Information for order

Item	Delivery form	Item no.
mikrozid® AF wipes premium softpack (50 wipes)	12/Carton	on request
mikrozid® AF wipes Jumbo tin (10 x 200 wipes)	10/Carton	on request
mikrozid® AF wipes tin (10 x 150 wipes)	10/Carton	on request
mikrozid® AF wipes Jumbo refill (20 x 200 wipes)	1/Carton	on request
mikrozid® AF wipes refill (20 x 150 wipes)	1/Carton	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Accessories

Accessories	Item no.
wall bracket Jumbo tub	134421
Wall mounting for the dispenser (small)	134417

Related Products

- mikrozid® PAA wipes
- mikrozid® sensitive wipes
- mikrozid® universal wipes

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com.

* The surface coverage of one wipe depends on the surrounding conditions and the structure of the wiped surface.



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

Air Liquide A company of the Air Liquide-Group

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 (0) 40 - 52100 - 0
Fax +49 (0) 40 - 52100 - 318
www.schuelke.com
mail@schuelke.com

Schülke & Mayr (Asia) Pte Ltd
Sime Darby Enterprise Centre
10 Jalan Kilang #04-04/05
Singapore 159410
Phone +65 6257 2388
Fax +65 6257 9388
www.schuelke.com.sg
mail.sg@schuelke.com

Schülke & Mayr (Asia) Sdn Bhd
Block A, Plaza Glomac
Lot A-06-01 & A-06-02
No 6 Jalan SS7/19, Kelana Jaya
47301 Petaling Jaya
Selangor, Malaysia
Phone +603 7885 8020
Fax +603 7885 8021
www.schuelke.com