

OPERATING INSTRUCTIONS



VENEN ENGEL 6 MOBIL

ENGLISH



PRELIMINARY REMARK

Congratulations on the purchase of this device! You will enjoy your Venen Engel very much! If you are satisfied with your Venen Engel, we would be very pleased if you submitted a review.

This guide includes details that should be taken into account by the user, to avoid risks, and to enable the safe use of the device.

Should you have any questions about its application, its accessories, or if you have suggestions, please do not hesitate to contact us! We are also happy to help you with any complications that develop during or after using the product.

Telephone: +49 (0) 666 17 48 91 05

Whatsapp: +49 1573 599 0084

Mail: service@venenengel.de

Web: www.venenengel.de

HelpDesk: help.venenengel.de

BEFORE YOU GET STARTED

- Please read the operating instructions carefully before using the device.
- Keep this user manual so you can consult it, if necessary.
- Venen Engel may not be used if you suffer from certain health conditions. Please read the list of contraindications carefully on last page.

STEP 1: BEFORE USE



CONNECT THE HOSE SET AND CUFFS

Connect the tubes to the connectors on the cuffs. Make sure the longest tube goes to the foot air chamber.

Note: We recommend that you do not detach the connectors after setup.



CONNECT THE TUBE SET AND THE CONTROL UNIT

There are two clips on the side of the multi-plug. Depress the clips then insert the multi-plug into the control unit.

Note: Please insert the small fittings of the multi-plug into the openings found above the air outlets.




CONNECT THE POWER SUPPLY TO THE CONTROL UNIT

Connect the power supply to both the control unit and an outlet to ensure power.

Note: To keep the device clean, we recommend wearing thin jogging pants or leggings. Keep pockets empty. Zip the cuffs completely closed to avoid sudden opening. Be careful not to kink the tubing during use.


STEP 2: OPERATION

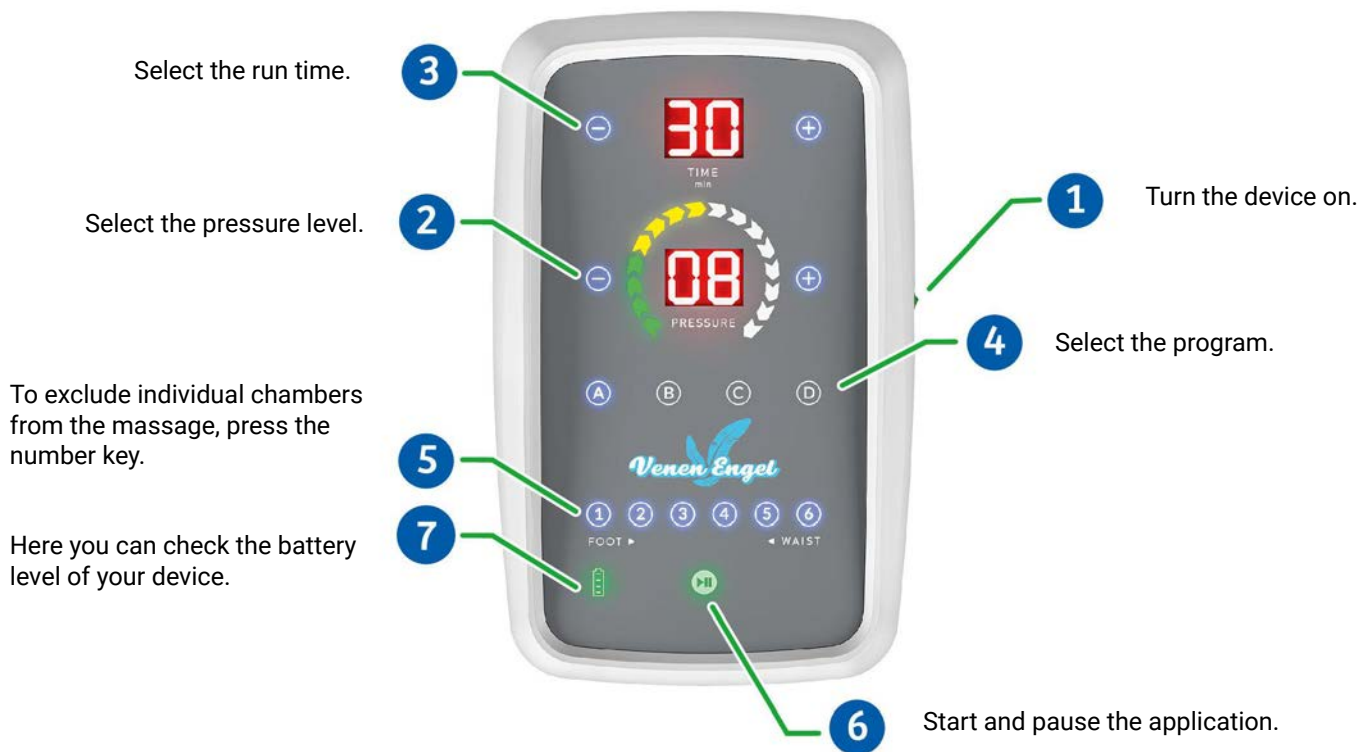

STEP 3: ENJOY THE MESSAGE

MESSAGE PROGRAMS A-D:

The air chambers in the leg cuffs are filled based on the selected program. This creates an optimal and relaxing gliding pressure message on both legs. Duration and pressure, in addition to optional additional functions can also be adjusted during the message.

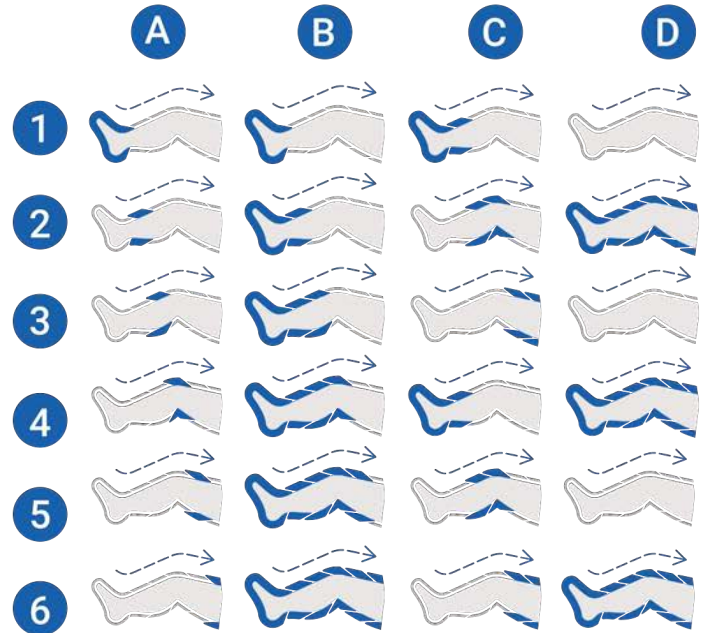

STEP 4: STOPPING AND DISASSEMBLY

After the set time has elapsed or after pressing the off switch, remove the multi-plug from the device. If you don't feel well during the message, you can switch the system off at any time.

Once turned off, you can open the cuffs. To quickly deflate the air chambers after use, press on the air chambers with both hands. The gray tube cuff connectors can be left permanently connected. For storage, place the cuffs (incl. tubes) on top of each other. Then roll them loosely together. The cuffs can be cleaned using a damp cloth and mild detergent. Allow the cuffs to dry thoroughly. Disinfectants such as Descosept pur or comparable products with a similar composition are suitable for disinfecting the cuffs.

TROUBLESHOOTING

AIR IS ESCAPING (AUDIBLY)

1. Check the tubes and connectors for damage or kinks that could cause air to be trapped.
2. Make sure the multi-plug is seated correctly in the control unit.

THE CUFFS INFLATE IN THE WRONG ORDER

1. Make sure the multi-plug is inserted in the control unit using the correct orientation. The multi-plug is seated correctly if the small fittings of the multi-plug are in the openings found above the air outlets.

I HAVE A LEG / ABDOMEN SET AND WISH TO USE ONLY THE LEG OR ABDOMEN CUFFS THIS TIME

If you would like to use your Vein Angel in a different configuration than the one that was shipped to you, you will need the appropriate hose set. All hose sets are available in our online shop. If you need help selecting a set, please contact us! We will be happy to help you.

THE DEVICE PUMPS AIR, BUT NEITHER OR ONLY ONE OF THE CUFFS IS INFLATING

If, after starting the programme, you find that not all the sections are filling with air, please check (1) if the multi-plug is correctly inserted into the controller. Also check that (2) the air hoses are not kinked and that you are not sitting on any of them. Finally, check (3) that the plugs in the cuffs are all properly inserted.


Instruction for Accessories

ATTACHING THE EXTENSION FLOP




PUTTING ON THE ARM CUFF

Step 1

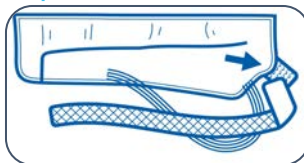

Step 2

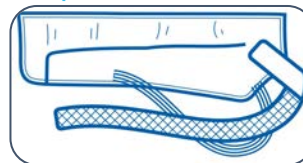

Velcro the strap to the back of the cuff.

Step 3



Step 4



Pull the cuff onto your arm and make sure the end is sitting on your shoulder. Run the strap under your other arm and attach it to the front of the cuff.

PLACEMENT OF THE ABDOMEN CUFF

The bottom of the cuff should sit directly against the top of the leg cuffs. **Tip:** put on the leg cuffs first. Then place the abdomen cuff behind you on the ground (inside facing up). Lay yourself back onto the cuff. Once you have adjusted it to the correct height, zip it around your stomach.

COMBINING PIECES

If you would like to use additional cuffs, make sure you pay attention to the number of air chambers in each one as you buy them. If you would like to use the additional cuffs in combination with the ones you have, you will need the appropriate hose set.

CONTENT & TECHNICAL DETAILS



Device




Power plug



Cuffs



Multi-Plug air hose set

Number of programmes	4
Number of air chambers	6
Weight [in kg]	-1,6
Power output [in W]	35
Mains frequency [in Hz]	50 / 60
Voltage [in V]	AC110-240
Timer setting [in min]	5-99
Pump output [in l/min]	12
Pressure [in mmHg]	30-240
Dimensions [in cm]	22 x 14 x 9
Protection class	II/ 
Temperature range [in °C] (use and storage)	10-40
Humidity [in %] (use and storage)	30-85
Battery:	
• Charging time / running time [in min]	130 / 180
• Type	Li-ion
• Model number	18650
• Voltage	11.1 V
• Capacity	5000 mAh
• Stored current	55.5 Wh

SAFETY INSTRUCTIONS

Keep these operating instructions in a safe place! Do not disassemble or burn the unit. Repairs to the control unit may only be carried out by specialists authorised by Wellcosan GmbH, otherwise the warranty may be voided. In case of damage to the unit, pull out the mains plug and switch off the unit. Also disconnect the mains plug and switch it off if the unit has come into contact with water or other liquids. Do not use the unit in damp rooms. Ensure that the unit is standing securely. Do not subject the device to shocks or throw them. The unit must not be covered by other objects during operation. Make sure that the hoses do not twist or kink during use and storage. Keep the unit away from sources of heat (e.g. direct sunlight, radiators and others). When you are not using the unit, the unit must be switched off. The unit switches itself off after 10 minutes of inactivity. When not in use also unplug the appliance from the mains. Always unplug the appliance from the mains before cleaning. Keep the product away from children or persons who have not been instructed in its use.



With the CE marking, the manufacturer or importer confirms the conformity of the product with the applicable EC directives and compliance with the "essential requirements" specified therein.



This marking on the unit means that you must not dispose of it in normal household waste at the end of its service life. Please have it disposed of by an electronic waste disposal company.



The abbreviation UKCA stands for UK Conformity Assessed. The marking is a prerequisite for the marketing of products within Great Britain.

